DICTUS INDOOR

An aid for people with Foot Drop
Art. No. DS-03; Dictus Indoor

Dear Customer,

Thank you for your purchase of the Dictus Indoor (DS-03) orthopedic aid. We hope that it will perform to your satisfaction. To avoid unnecessary problems and trouble, please read the instructions below carefully before using the orthopedic aid.

If any part of your Dictus Indoor (DS-03) should require servicing, please contact your retailer.

INSTRUCTION

Use

The Dictus Band (DS-02) is an orthopedic aid, mainly intended for people suffering from Foot Drop. The active part is a latex elastic band, which lifts the front of the foot in an upward position.

The Dictus Indoor (DS-03) is a complement to and is used in conjunction with Dictus Band (DS-02). Dictus Indoor (DS-03) provides support when the user is moving about indoors or without shoes.

Parts

Dictus Indoor (DS-03)

- Foot strap consisting of a leather band with adjustment straps and mounting hooks.

Mounting

The foot strap is placed on top of the user's sock or stocking (not directly on skin).

The opening and tightening function is managed with two hook and loop straps. The latex elastic band is tied to the D ring on the Dictus Band (DS-02) then connected to the plastic hooks on the top of the Dictus Indoor (DS-03).

Precautions and Care Advice

Before and during use of the Dictus Indoor (DS-03), always be sure that the foot strap is correctly positioned and that the hook and loop fasteners and plastic hooks are in good condition.

The Dictus Indoor (DS-03) foot strap must be cleaned regularly with a damp cloth. No detergent is to be used.

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WARNING!

There is a risk of falling if the latex elastic band or any other part breaks or comes loose, causing the foot to no longer stay in an upward position.

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Warranty

The Dictus Indoor (DS-03) comes with a 1 (one) year warranty by Erimed International KB. The warranty is granted only if the user has followed the instructions and care advice and not exposed the Dictus Indoor to exceptional strain.

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PLEASE OBSERVE

The Dictus Indoor package includes small parts which should be kept away from small children.

This product is not a toy.

Dictus Indoor must not be used for purposes other than those described in this product's operation instructions.

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