



ALLIED
MEDICAL
LIMITED



Quick Reference Guide for HS-268 Auto-folding Function

- When unfolding the scooter ensure the scooter is laying flat with the rear wheels on the ground. Ensure this is on a flat surface with no obstructions and all packaging is removed.
- Underneath the seat, there are two buttons with lights showing the scooter folded and unfolded. If the orange lights are lit this indicates it is ready to fold or unfold.
- Use the remote and click the "On" button. The scooter will automatically unfold.
- If the scooter does not unfold fully, the sensors have detected an obstacle that could pose a problem for unfolding. Remove any obstructions and try again.
- Once it has unfolded, simply lift the seat back up into ready position.
- To fold up the scooter, turn off the scooter, put the seat back down and click "Off" on the remote.
- Once folded, you can lift the scooter up onto its front wheels and push it into storage
- You can also use the two buttons under the seat to fold and unfold without the use of the remote.
- If the scooter has been idle for some time the light will not show. You can wake the scooter up by turning it on and off again.
- The scooter will not fold or unfold in its upright position.
- To see how it folds, watch our video online [here](#).